**DE**

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Seite 2

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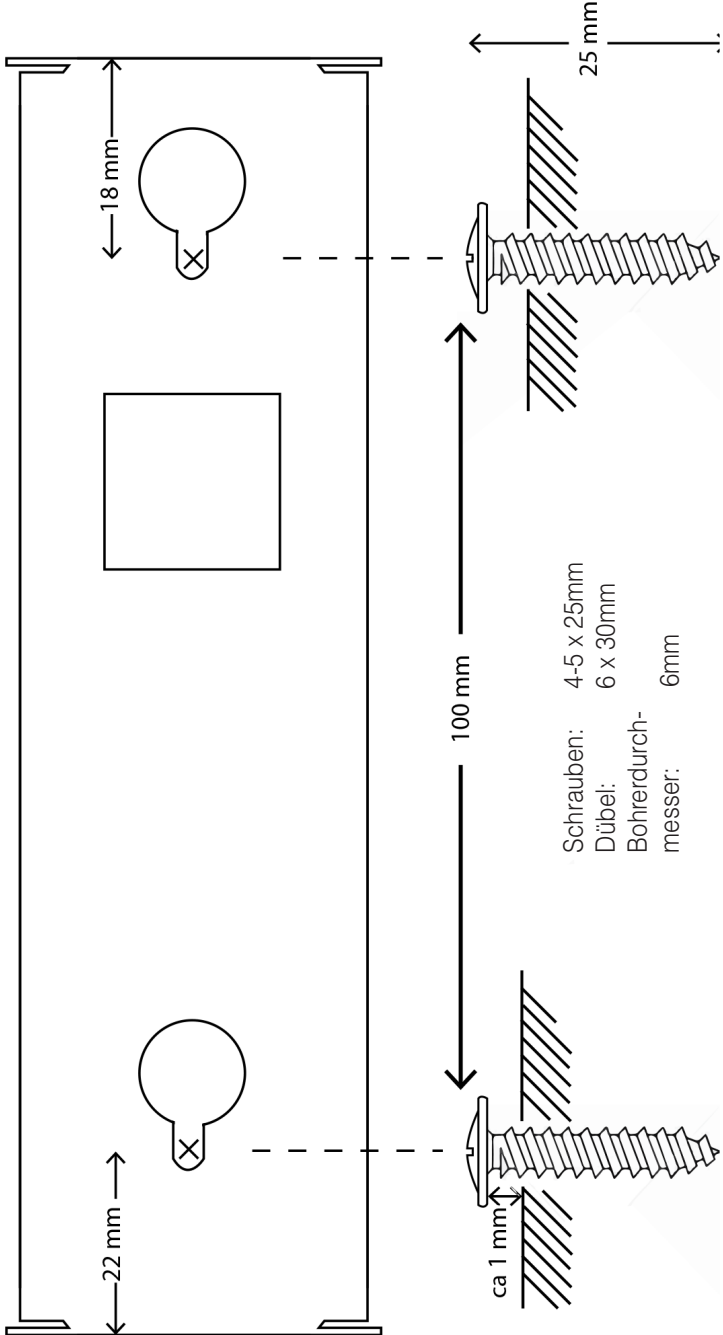
# Bohrschablone

## Content

User Manual for »T-sign«	
About »T-sign« .....	11
Package contents.....	11
Set up/ Installation instructions for T-sign .....	12
Normal operation mode .....	12
Installation (refer to the enclosed Quick Guide) .....	12
Calibration mode .....	13
DIP-switch configuration .....	13
Program characteristics .....	14
Program 1 .....	14
Program 2 .....	14
Maintenance and care .....	15
Warranty .....	15
Disposal .....	15
Specifications .....	16
Drilling template .....	17

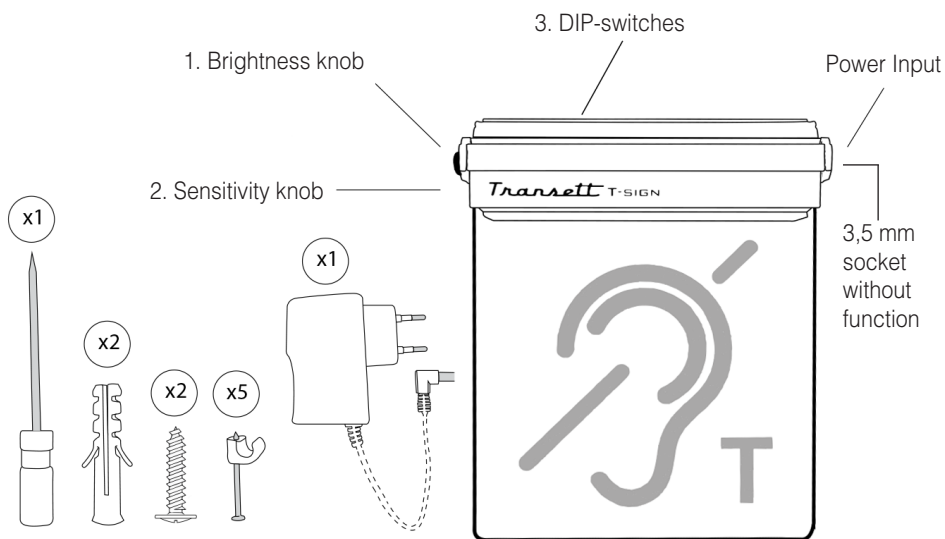
## About »T-sign«

»T-sign«, the hearing loop indicator, is designed to make the audience aware of an active hearing loop. »T-sign« is programmed with various modes of indication that accordingly reflect how well the sound level is transmitted via the loop.

## Package contents

Please check if all the listed parts are included:

- »T-sign«
- 1 power supply unit
- 5 pcs. cable clamps
- 2 pcs. screws
- 2 pcs. wall plugs
- 1 screwdriver
- User manual
- Warranty card

Should any parts be missing, please contact your dealer or the manufacturer directly.

## Set up/ Installation instructions for »T-sign«

### Normal operation mode

In order to run »T-sign« in normal operation mode, just start it up by plugging the power supply in and let it run its startup sequence for approx. five seconds.

During the startup sequence, the »T-sign« indicates which program it runs by the number of visible of green flashes.

Program one shows one flash and program two shows two flashes (see page 14, program characteristics).

No signal:	»T-sign« panel is dark
Weak signal:	Blinking slowly in green
Normal signal:	Steady green light
Strong signal:	Steady red light

### Installation (Refer to the enclosed Quick Guide)

1. Preparations before set-up: A functioning inductive hearing loop, set up in accordance with the SIS 60118-4 standard and a device providing a sound source to the hearing loop, e.g. a microphone or a TV.

2. Choose an appropriate position for the »T-sign«. It is not recommended to set up »T-sign« close to electronic devices due to signal interference.

3. Connect the power supply to the »T-sign« and keep it at or nearby the chosen mounting place.

Make sure that the hearing loop is not in use. Verify that there is no other signal interference that affects the »T-sign«. The »T-sign« remains dark when there is no detectable signal in the hearing loop.

Depending on the magnetic field strength of the hearing loop at the chosen location, the corresponding sensitivity needs to be set on the »T-sign«. A high sensitivity setting in combination with background signal interference might cause the T-sign to show false indications. If this is the case, consider choosing a different location with stronger field strength and/or less signal interference from its surroundings.

4. Set two screws into the wall according to the enclosed drilling template (page 17).

5. Fine-tune the sensitivity of »T-sign« with the included screwdriver. Follow the instructions in the next paragraph "Calibration mode".

## Calibration mode

1. Turn off »T-sign« by unplugging the DC connector or disconnecting the power supply from the mains electricity.

Wait a few seconds until it has shut down.

2. Turn the light intensity knob to minimum.

3. Turn on »T-sign« by plugging the DC connector in or by reconnecting the power supply.

4. After the initial one- or two green flashes (depending on program selection), turn the light intensity knob of the »T-sign« to maximum. This must be done within two seconds after the green flash(es) are seen.

5. »T-sign« now indicates that it is in calibration mode by two short green double-flashes.

6. When in calibration mode, the »T-sign« shows the magnetic field strength in real time where it is mounted. The calibration mode is used to adjust the sensitivity, so that the »T-sign« changes color from green to red when the magnetic field strength at its location reaches 400 mA/m at 1 kHz.

If you don't have access to a measuring device, adjust the sensitivity while sending a strong signal to the hearing loop (as received at a normal listening position within the audience area) so that the »T-sign« turns red when the transmission is loud.

Depending on the setting, the »T-sign« will be more or less prone to change to red when a strong signal is received.

7. Finally, disconnect the power supply, wait approximately five seconds and then reconnect the power supply again.

## DIP-switch configuration

- DIP-sw 1: Automatic control of light intensity related to environmental light (off/on)
- DIP-sw 2: Reduced sensitivity for weak signals under 3 dB (off/on). This can be a helpful feature when high sensitivity is needed in combination with some interference in the vicinity.
- DIP-sw 3: Change indication of strong signal from a steady red light to flashing red light (off/on)
- DIP-sw 4: Program selection 1 & 2

## Program characteristics

### Program 1:

When set to Program 1, the »T-sign« will react relatively quickly to changes in the magnetic field strength of a present loop. This can be of advantage e.g. in a teaching situation where the speaker is interested in knowing that the correct microphone settings are used.

- From dark to some indication: 1 sec
- From weak to normal signal indication: 2 sec
- From normal to strong signal indication: 4 sec
- From strong to normal signal indication: 2 sec
- From normal to weak signal indication: 4 sec
- Time to dark »T-sign« when no signal detected: 3 sec

### Program 2:

In Program 2, the »T-sign« reacts more slowly to changes in magnetic field strength. This is practical e.g. in situations where you don't have the possibility to directly adjust the level of the outgoing signal to the hearing loop. It is also useful for indicating that the hearing loop is functioning at a basic level.

- From dark to some indication: 5 sec
- From weak to normal signal indication: 4 sec
- From normal to strong signal indication: 15-25 sec
- From strong to normal signal indication: 2 sec
- From normal to weak signal indication: 25 sec
- Time to dark T-sign when no signal detected: 60 sec.

The time values stated above are approximations and based on a 1 kHz signal with a 3 dB step below or above the threshold.

When using »T-sign« for speech, the time values will vary depending on the characteristics and intensity of the signal.

## Maintenance and care

The »T-sign« does not require any maintenance. If the unit becomes dirty, simply wipe it clean with a soft, damp cloth. Note: Disconnect the »T-sign« from the power supply first. Never use spirits, thinners or other organic solvents. Do not place the »T-sign« where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

## Warranty

The »T-sign« is a very reliable product. Should a malfunction occur despite the unit being set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge.

If possible send the product back in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the product seal). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

## Specifications

Input power: 15V, 1A via external power supply 110 – 230V AC  
Power consumption: 1 W

Sensitivity, transition from normal to strong indication (steady green to red color light)

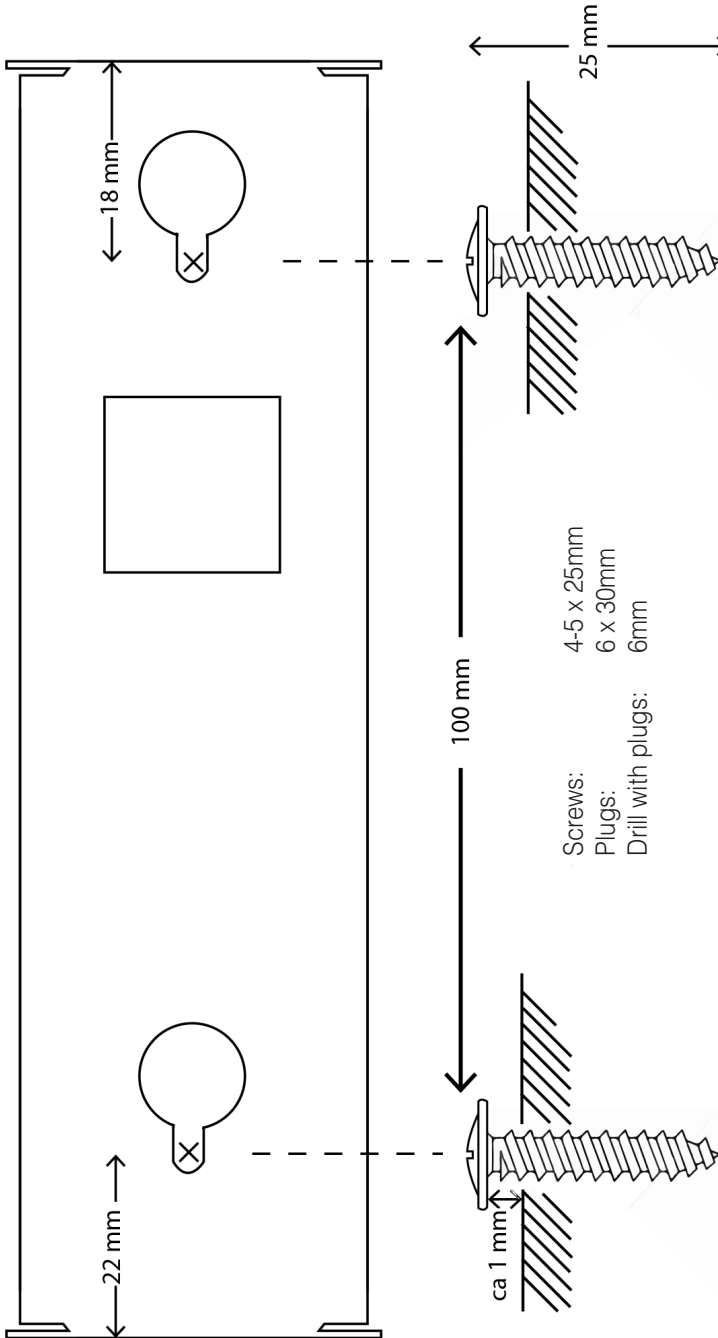
- Sensitivity knob in min position: + 9 dB signal (1 kHz, ref 400 mA/m)
- Sensitivity knob in max position: - 22 dB signal (1 kHz, ref 400 mA/m)
- Frequency range: 300 Hz – 2000 Hz (rel -3 dB)

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**Sarabec Ltd**